



"Fast & without anaesthesia, FibroScan® makes the monitoring of my disease more convenient."


What is FibroScan?

Examination with FibroScan®, also called transient elastography, is a technique used to assess liver stiffness (measured in kPa correlated to fibrosis) without invasive investigation.

The result is immediate, it shows the condition of the liver and allows physicians to diagnose and monitor disease evolution in conjunction with treatment and collateral factors.

Exam results help to anticipate various complications, as well as to monitor and assess the damage caused by conditions such as cirrhosis.

LIVER STIFFNESS DETERMINES THE PATHOLOGICAL STATE

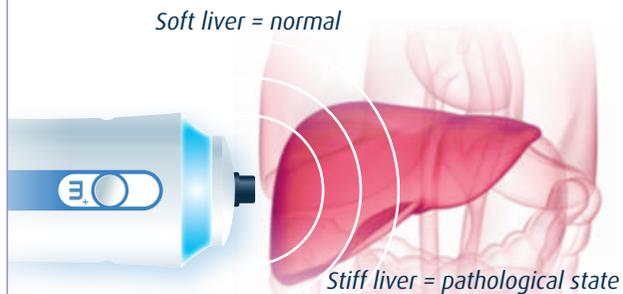

FibroScan® measures liver stiffness that may be subject to variation due to inflammation and fibrosis stage⁽²⁾.

Is it painful? No.

The FibroScan examination is painless, quick and easy. During measurement, you feel a slight vibration on the skin at the tip of the probe.

What does the FibroScan examination consist of?

- You lie on your back, with your right arm raised behind your head. Physician applies a water-based gel to the skin and places the probe with a slight pressure.
- The examination includes 10 consecutive measurements made at the same location.
- The result is delivered at the end of the examination, it's a number which can vary from 1.5 to 75 kPa. Your doctor will interpret the result.

What does the result mean?

Your physician interprets the result according to your history and underlying disease.



"When my physician suggested me to try the FibroScan®, I accepted immediately."

In brief

Result obtained with FibroScan® will allow your physician to monitor your liver health over time with or without treatment.

Recommendations for use


Before performing liver stiffness measurement using the FibroScan, it is recommended to wait 2 or 3 hours after eating.

Who can prescribe the FibroScan® examination?

Your physician or hepatologist will indicate the most appropriate time for you to have the examination.

To learn more

Ask your physician or hepatologist.

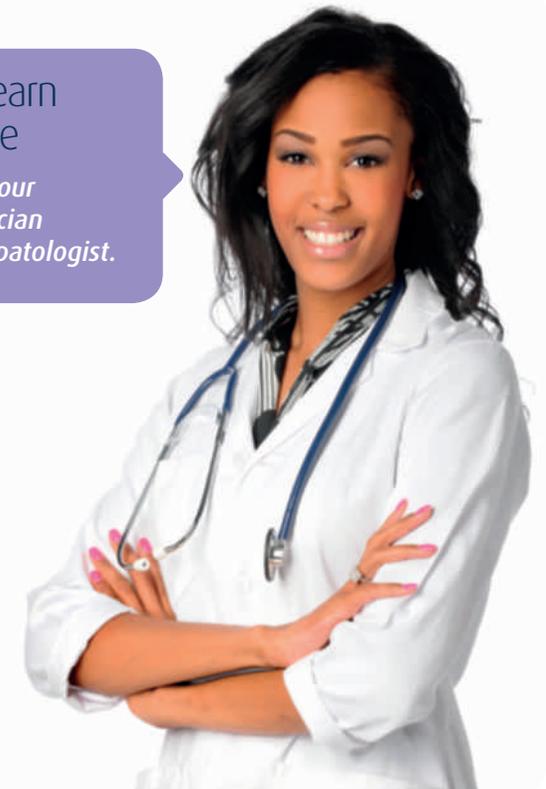



"For my daughter, I chose a painless examination: the FibroScan®."

PATIENTINFORMATIONEN012016- Revision date 012016- FibroScan® is a medical device manufactured by Echosens. This medical device is a regulated health product which, according to this regulation, bears the CE mark. FibroScan® is indicated for the non invasive measurement of liver stiffness (E) and controlled attenuation parameter (CAP) in human beings.

It is expressly recommended to closely read the instruction of the users' guide and labeling of the device. FibroScan® examination must be performed only by operator certified by the manufacturer or its accredited local representative. The values obtained with FibroScan® must be interpreted by a physician experienced in dealing with liver disease, taking into account the complete medical record of the patients.

1. Melin, P., et al., Dépistage non invasif de la fibrose : Intérêt du FibroScan® en consultation d'alcoologie [Noninvasive screening of fibrosis: interest of FibroScan® in alcohol addiction consultation]. *Alcoologie et Addictologie*, 2005;27:191-196.
2. Mueller, S. and L. Sandrin, Liver stiffness: a novel parameter for the diagnosis of liver disease. *Hepatic Medicine: Evidence and Research*, 2010: p. 49-67

PATIENT INFORMATION
FIBROSCAN® EXAMINATION



Rapid, painless, and completely non-invasive examination of your liver...

Have you ever heard about FibroScan?

What difference does FibroScan make for me?

- FibroScan® provides immediate results, it's easy and fast (5-10 minutes).
- The exam is painless and non-invasive.
- In case of close follow-up, the examination can be safely repeated.



FibroScan probes are adapted to all body-types patients.



100% of patients accept a FibroScan examination⁽¹⁾



FibroScan®
BY ECHOSENS
MORE ABOUT : WWW.ECHOSENS.COM



FLASH THIS CODE

CE 0459 - ISO 13485 - ECHOSENS AND FIBROSCAN® ARE REGISTERED TRADEMARKS © COPYRIGHT ECHOSENS ALL RIGHTS RESERVED - PATIENTINFORMATIONEN012016

FibroScan®